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**EN** Close all fasteners according to instructions.

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### MANUFACTURER INFORMATION

Instructions for use  
Ready for use foot bandage for **Pedilay® med** issued 16.02.2021 by Pedilay® Care GmbH. Version 3.0 / 02.2021

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### Local court Munich Registration number: HRB 10314

Sales tax identification number according to §27 sales tax law: DE299377700

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### WHAT IS PEDILAY® MED? WHAT IS IT USED FOR?

**Pedilay® med** is an adaptable and ready for use foot bandage that encloses the foot up to the ankle and supports medical treatments. **Pedilay® med** acts as a biocompatible bandage layer that protects and supports the skin. **Pedilay® med** can be used on its own or in combination with other products for medical treatment.

When used as a secondary or tertiary bandage on injured skin, **Pedilay® med** acts as an additional bandage layer. **Pedilay® med** can help protect the layers underneath and absorb excess fluids.

**Pedilay® med** can be used in patients receiving short- or long-term medical foot treatment or who are bedridden or have limited mobility.

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**Pedilay® med** must not be used: Directly on injured skin or open wounds without a primary layer underneath (e.g. plaster)

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If there is any uncertainty about how to use it or if your health deteriorates, use should be suspended immediately and medical advice should be sought. Please report all serious incidents to **Pedilay® Care GmbH** and the **Federal Institute for Drugs and Medical Devices (BfArM)**: Kurt-Georg-Kiesinger-Allee 3, 53175 Bonn.

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**Please note:**  
Non-medical users should try on and select the correct size in consultation with a healthcare professional.

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Shoe sizes and foot measurements serve as guidance only when selecting **Pedilay® med** sizes. Other factors may require a different size in individual cases. For size selection, see also the next page.

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**Pedilay® med** in the sizes S and L are based on a normal foot shape in accordance with your shoe size and foot length. For feet with special requirements, such as severe deformation or increase in volume, **Pedilay® med XL** may be recommended.

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### SIZE SELECTION

The following factors may require you to select **Pedilay® med** in a different size: Skeletal deformities, extremely voluminous feet, swollen feet, amputations, additional primary/secondary layers and additional bandages, extremely voluminous feet due to oedema or diabetic foot syndrome, for example

### CORRECT FIT

**Pedilay® med** is correctly applied when there are no cavities, no holes or folds and the material does not produce any pressure/compression

### SIGNS OF AN INCORRECT FIT

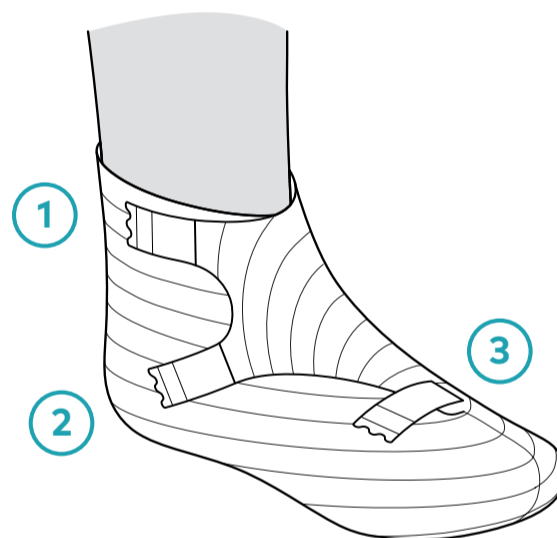
**Pedilay® med** feels tight or causes a feeling of numbness. The material has lost its wave structure or appears transparent. Seams or elastic fibres cause permanent constrictions. The fasteners cannot be closed.

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
### FASTENERS

- 1 **ANKLE**  
Runs parallel to the border
- 2 **BACK OF THE FOOT**  
Runs along the back of the foot (instep) diagonally towards the heel
- 3 **FOREFOOT**  
Runs along the forefoot (only in sizes S and L)

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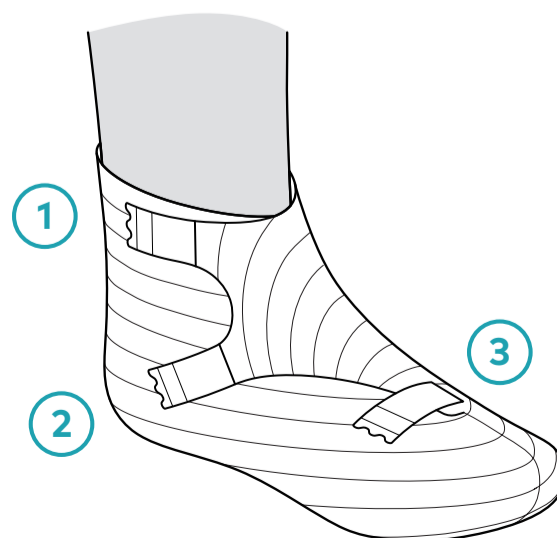### HYGIENE AND DISPOSAL

For optimum hygiene, change **Pedilay® med**:

- Daily in the case of acute, infectious and weeping wounds
- When the material is stretched or torn
- When the material is dirty or soaked
- After 3 days at the latest

Dispose of **Pedilay® med** with ordinary household waste immediately after taking it off.

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Special precautions must be taken with infectious wounds:

- Use disposable gloves or disinfect your hands before and after use
- Dispose of **Pedilay® med** in a closable waste container
- In inpatient facilities, the relevant company and legal regulations also apply

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